INSTRUCTIONS FOR SALIVA COLLECTION

Using the SimplOFy® Saliva DNA Collection Device


NEW
iDEAS
Brain Imaging Study
for Memory Loss

alzheimer's
association

- 1 | Do not eat, drink, smoke, or use oral hygiene products for at least 10 minutes before you start the collection process.
- 2 | Rinse the mouth with water and discard. Please wait at least 5 minutes after this rinse to start the collection procedure.
- 3 | After the five minutes have passed, follow the illustrated instructions below. Video instructions are available on the website: <https://4saliva.com/products/simplofy/>


1

Open the package containing the SimplOFy® Saliva DNA Collection Device. Place contents on a clean and dry surface.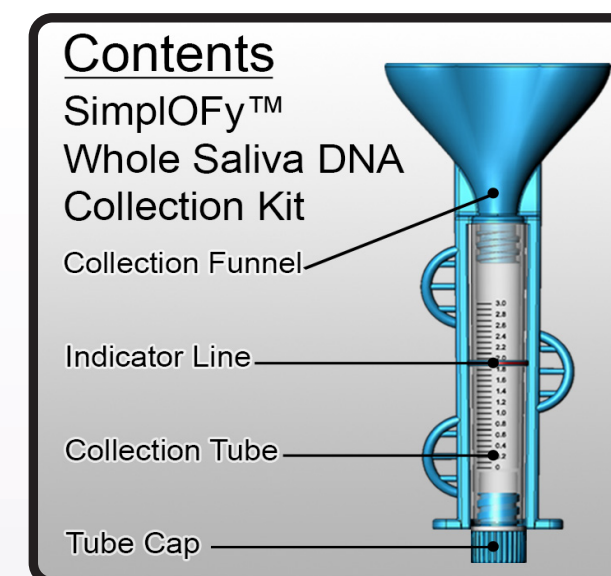


2

Place lips over the opening of the Collection Tube Funnel. Collect saliva until the sample level reaches the red Indicator Line.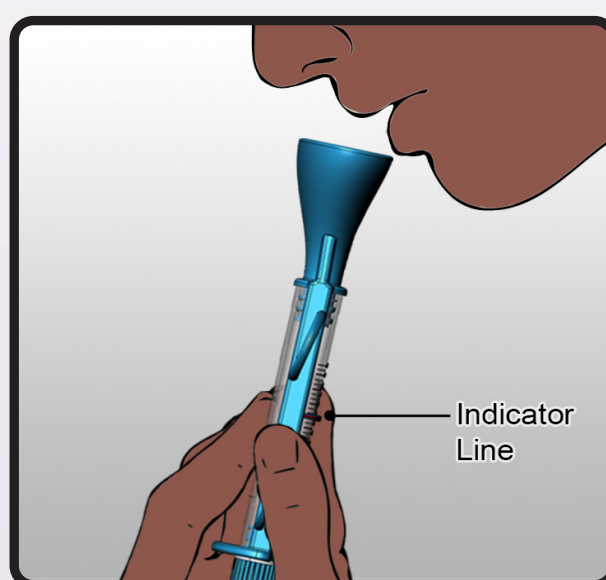


3

When collection has been completed, unscrew the Funnel from the Collection Tube by rotating the tube in the direction shown until the tube is completely detached from the Funnel.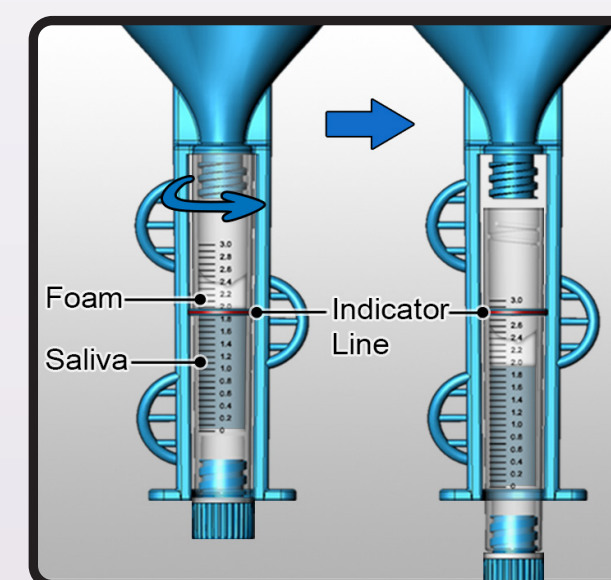


4

Remove the Cap from the bottom of the Collection Tube and screw it tightly closed.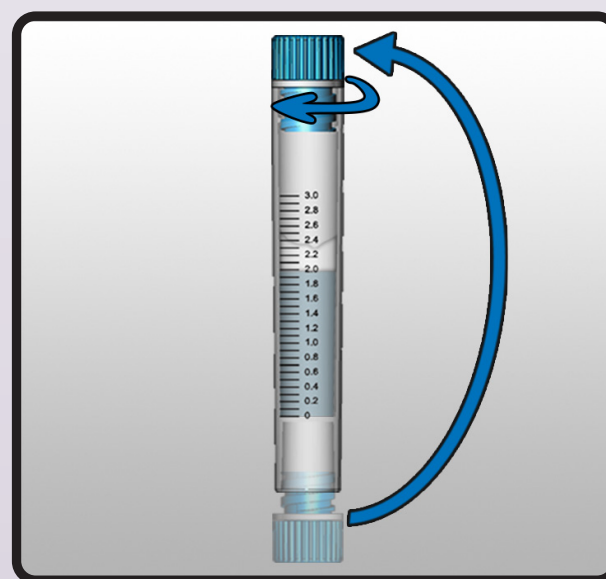

5


Vigorously invert the tube for 1 minute to mix the sample. The sample is now ready for shipment.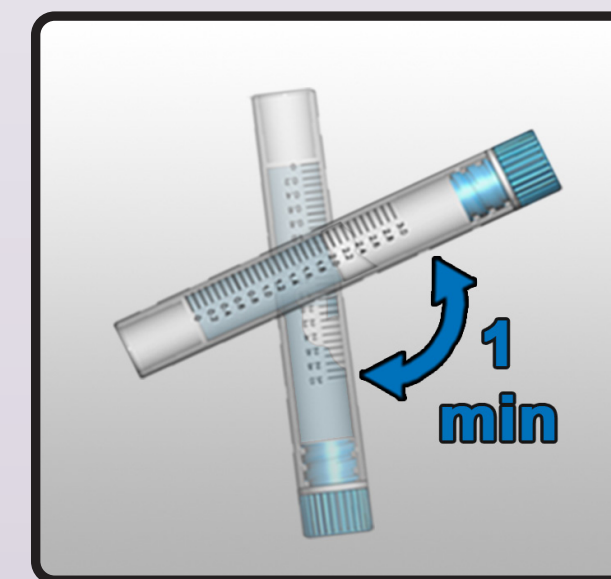


6

Saliva Shipment


Follow instructions to return samples to the Biomarker Lab.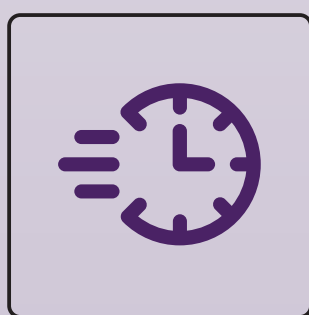


1 | Your sample is stable once it is mixed with the DNA stabilization buffer, but it is strongly recommended to ship the sample at your earliest convenience.

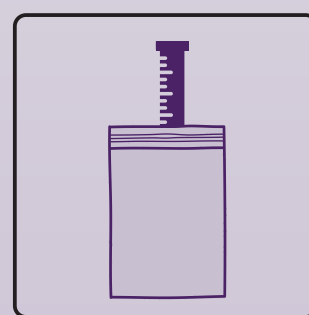
2 | Place the collection tube containing your sample in the plastic specimen bag.

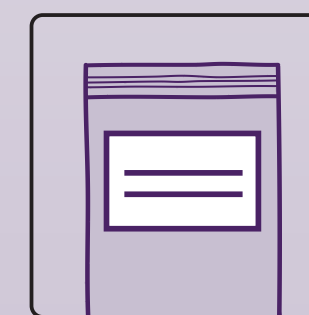3 | Complete the 'Collection Date' and 'Collection Time' fields located on Sample Label stuck to the outside of the specimen bag.

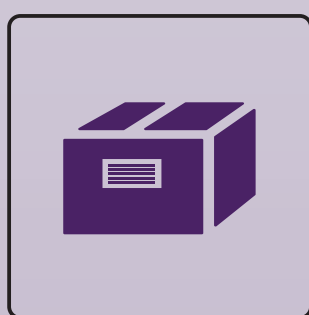4 | Place the labeled specimen bag into the provided pre-paid shipping box and seal it

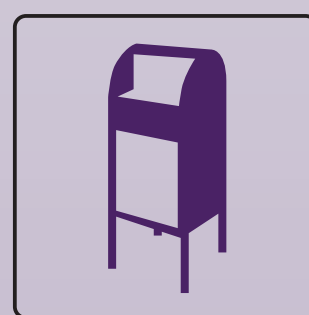5 | Samples can be dropped off at your nearest local post mailbox.

6 | If you have any questions or concerns regarding collection or shipment of samples, please contact newideas-participant@alz.org or 800.272.3900.

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