

ACR Digital Mammography Phantom Scoring Key*

Test Object	Full Point	Half Point
Fibers (6)	<ul style="list-style-type: none"> • Full length visible (≥ 8 mm long) • Correct location • Correct orientation • 1 break allowed (must be \leq width of fiber) 	<ul style="list-style-type: none"> • At least half of length visible (≥ 5 and < 8 mm long) • Correct location • Correct orientation • 1 break allowed (must be \leq width of fiber)
Speck Groups (6)	<ul style="list-style-type: none"> • 4 - 6 specks visible • Correct locations 	<ul style="list-style-type: none"> • 2 - 3 specks visible • Correct locations
Masses (6)	<ul style="list-style-type: none"> • Density difference visible • Border is continuous and generally circular ($\geq 3/4$ border visible) • Correct location 	<ul style="list-style-type: none"> • Density difference visible • Border is not continuous or generally circular ($\geq 1/2$ and $< 3/4$ border visible) • Correct location
Artifacts	Only fail for artifacts if they are in a location that could impact clinical interpretation and they are clinically significant. Fail if: <ul style="list-style-type: none"> • Artifacts are as prominent as (or more prominent than) the visible test objects in the phantom image, or • Artifacts obscure test objects in the phantom, or • Artifacts could affect clinical interpretation 	

Performance Criteria: ≥ 2 fibers, ≥ 3 speck groups, ≥ 2 masses, no clinically-significant artifacts

* Consult the ACR 2018 Digital Mammography Quality Control Manual and the [FAQs](#) for complete information on scoring the phantom.

